
# SPIRO-BALL SMILE LUNGENTRAINER

## ANLEITUNG

Art.-Nr.  03393 · Kategorie:  Atemtrainer

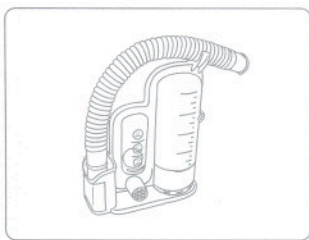# Spiro-Ball

REF 259 - 13000

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**Instructions:** (A) Connect the open end of the tube into the hole. (B) Slide the yellow level-indicator to the volume (mL) prescribed by your doctor. (C1) Right before beginning the exercise, completely exhale all the air from your lungs. (C2) Place the mouthpiece in your mouth and hold it firmly between your lips to keep air from escaping when inhaling. (C3) Inhale slowly and deeply with the device in the vertical position, keeping the flow-indicator ball in the happy-face position. Inhale as deeply as possible. When you are unable to inhale any more, hold your breath a few seconds and check how high the piston reached. Try to reach your prescribed target level. Repeat the procedure the prescribed number of times. Record your progress curve on the attached chart.

**Warnings:**

- For hygiene purposes, wash the mouthpiece before and after use and keep it in the special slot when not in use; return the Spiro-Ball to its original packing until you use it again.
- Do not exhale into the device. Humidity can cause it to malfunction and create conditions that favor the growth of infectious organisms.
- Do not keep the device in a humid place.
- Keep the device in the vertical position during the exercise.
- The air-intake holes should remain open during use.

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**Description:** The Spiro-Ball is a product for use in respiratory therapy. The system is made up of a mouthpiece, an inhaling tube and the unit itself containing three moveable parts: a flow-indicator ball, a piston that measures the volume inhaled and an indicator that shows the target level to be reached.

**Indications:** Deep-breathing exercises are an important part of your recuperation. Using this product helps patients return to their normal breathing patterns more quickly and decreases the risk of respiratory complications.

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**Sport-Tec GmbH**