	cobas click kapillärhelblod	cobas click EDTA-helblod	cobas b 101 kapillärhelblod	cobas b 101 EDTA-helblod
Avvikelse (%) vid 5.7 % (39 mmol/mol) HbA1c	2.0	1.4	1.9	1.3
Avvikelse (%) vid 6.5 % (48 mmol/mol) HbA1c	1.1	0.6	0.9	0.6
Avvikelse (%) vid 8.0 % (64 mmol/mol) HbA1c	-0.1	-0.3	-0.4	-0.5
Avvikelse (%) vid 12 % (108 mmol/mol) HbA1c	-1.8	-1.7	-2.2	-1.9

Ytterligare information

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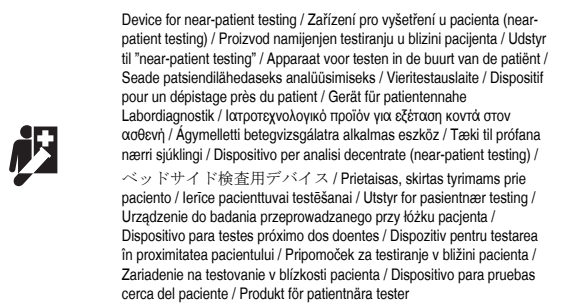
References / Reference / Reference / Referencer / Literatuurverwijzingen / Viited / Viitteen / Références bibliographiques / Literatur / Βιβλιογραφία / Ιστολομiejzyżek / Bibliografía / Letteratura / 参考文献 / Nuorodos / Atsauces / Referanser / Písmennictwo / Bibliografia / Refernte / Referência / Literatura / Referencias bibliográficas / Referencer
1 So JCC, Ma ESK. Hemoglobin and hemoglobinopathies. In: Rifai N, Chiu RWK, Young I, Burnham CAD, Wittwer CT, editors. Tietz Textbook of Laboratory Medicine, Saunders Elsevier, Philadelphia, 7th edition, 2023, chapter 77, p. 1039-1077.e5.
2 Sacks DB. Diabetes mellitus. In: Rifai N, Chiu RWK, Young I, Burnham CAD, Wittwer CT, editors. Tietz Textbook of Laboratory Medicine, Saunders Elsevier, Philadelphia, 7th edition, 2023, chapter 47, p. 502-543.e12.
3 Goldstein DE, Little RR, Lorenz RA, et al. Tests of glycemia in diabetes. Diabetes Care. 2004 Jul;27(7):1761-1773.
4 Sherwani SI, Khan HA, Ekhzaimy A, et al. Significance of HbA1c Test in Diagnosis and Prognosis of Diabetic Patients. Biomark Insights 2016 Jul 3;11:95-104.
5 International Diabetes Federation Guideline Development Group. Global guideline for type 2 diabetes. Diabetes Res Clin Pract 2014 Apr;104(1):1-52.
6 American Diabetes Association Professional Practice Committee. 6. Glycemic Goals and Hypoglycemia: Standards of Care in Diabetes-2024. Diabetes Care. 2024 Jan 1;47(Suppl 1):S111-S125.
7 American Diabetes Association Professional Practice Committee. 2. Diagnosis and Classification of Diabetes: Standards of Care in Diabetes-2024. Diabetes Care. 2024 Jan 1;47(Suppl 1):S20-S42.
8 LeRoith D, Biessels GJ, Braithwaite SS, et al. Treatment of Diabetes in Older Adults: An Endocrine Society* Clinical Practice Guideline. J Clin Endocrinol Metab 2019 May 1;104(5):1520-1574.
9 Besser REJ, Bell KJ, Couper JJ, et al. ISPAD Clinical Practice Consensus Guidelines 2022: Stages of type 1 diabetes in children and adolescents. Pediatr Diabetes 2022 Dec;23(8):1175-1187.
10 Shah AS, Zeitler PS, Wong J, et al. ISPAD Clinical Practice Consensus Guidelines 2022: Type 2 diabetes in children and adolescents. Pediatr Diabetes. 2022 Nov;23(7):872-902.
11 Kobold U, Jeppsson JO, Duellfer T, et al. Candidate reference methods for hemoglobin A1c based on peptide mapping. Clin Chem 1997;43:1944-1951.
12 Jeppsson JO, Kobold U, Frike A, et al. Approved IFCC reference method for the measurement of HbA1c in human blood. Clin Chem Lab Med 2002;40:78-89.
13 Consensus Statement on the Worldwide Standardization of the Hemoglobin A1c Measurement. American Diabetes Association, European Association for the Study of Diabetes, International Federation of Clinical Chemistry and Laboratory Medicine and International Diabetes Federation Consensus Committee. Diabetes Care 2007;30:2399-2400.
14 Bergenstal RM, Beck RW, Close KL, et al. Glucose ManagementIndicator (GMI): A New Term for Estimating A1C From Continuous Glucose Monitoring. Diabetes Care 2018 Nov;41(11):2275-2280.
15 Beck RW, Bergenstal RM, Cheng P, et al. The Relationships Between Time in Range, Hyperglycemia Metrics, and HbA1c. J Diabetes Sci Technol 2019 Jul;13(4):614-626.
16 Hirsch IB, Welsh JB, Calhoun P, et al. Associations between HbA1c and continuous glucose monitoring-derived glycaemic variables. Diabet Med 2019 Dec;36(12):1637-1642.
17 David M, Nathan, MD1, Judith Kuenen, MD2, Rikke Borg, et al. Translating the A1C Assay Into Estimated Average Glucose Values. 2008 Aug;31(8):1473-8. doi: 10.2337/do08-0545.
18 Nathan DM, Kuenen J, Borg R, et al. Translating the A1C assay into estimated average glucose values. Diabetes Care 2008;31:1473-1478.
19 Bunn HF, Gabbay KH, Gallop PM. The glycosylation of hemoglobin: relevance to diabetes mellitus. Science 1978;200:21-27.
20 American Diabetes Association: eAG/A1C Conversion Calculator https://professional.diabetes.org/diagloglucose_calc.
21 WHO. Use of glycated haemoglobin (HbA1c) in the diagnosis of diabetes mellitus: abbreviated report of a WHO consultation. World Health Organization, Geneva. , WHO Guidelines Approved by the Guidelines Review Committee. WHO/NMH/CHP/CPM/11.1. 2011:1-25.
22 American Diabetes Association: Standards of medical care in diabetes - 2019. The Journal of clinical and applied research and education. January 2019; 42(Supplement 1):S1-S193.
23 Miedema K. Influence of hemoglobin variants on the determination of glycated hemoglobin. Klin Lab 1993;39:1029-1032.
24 Chang J, Hoke C, Ettinger B, et al. Evaluation and Interference Study of Hemoglobin A1c Measured by Turbidimetric Inhibition Immunoassay. Am J Clin Pathol 1998;109(3):274-278.
25 Rohlfing C, Connolly S, Englund J, et al. Effect of elevated fetal hemoglobin on HbA1c measurements: four common assay methods compared to the IFCC reference method. Clin Chem 2006;52 Suppl 6:A108.
26 International Expert Committee Report on the Role of the A1C Assay in the Diagnosis of Diabetes. Diabetes Care 2009;32(7):1327-1334.
27 Sacks DB, Bruns DE, Goldstein DE, Maclaren NK, McDonald JM, Parrott M. Guidelines and recommendations for laboratory analysis in the diagnosis and management of diabetes mellitus. Clin Chem. Mar 2002;48(3):436-472.
28 Behan KJ, Storey NM, Lee HK. Reporting variant hemoglobins discovered during hemoglobin A1c analysis - common practices in clinical laboratories. Clin Chim Acta. Aug 2009;406(1-2):124-128.
29 Breuer J. Report on the Symposium "Drug effects in Clinical Chemistry Methods". Eur J Clin Chem Clin Biochem 1996;34:385-388.
30 Sonntag O, Scholer A. Drug interference in clinical chemistry: recommendation of drugs and their concentrations to be used in drug interference studies. Ann Clin Biochem 2001;38:376-385.
31 American Diabetes Association: 1. Improving Care and Promoting Health in Populations: Standards of Medical Care in Diabetes-2020. Diabetes Care. 2020;43(Suppl 1):S7-S13.
32 American Diabetes Association: 2. Classification and Diagnosis of Diabetes: Standards of Medical Care in Diabetes-2020. Diabetes Care. 2020;43(Suppl 1):S14-S31.

- 33 The Diabetes Control and Complications Trial Research Group. The effect of intensive treatment of diabetes on the development and progression of long-term complications in insulin-dependent diabetes mellitus. N Engl J Med 1993;329:977-986.
34 Brown A, Reynolds LR, Bruemmer D. Intensive glycemic control and cardiovascular disease: an update. Nat Rev Cardiol. Jul 2010;7(7):369-375.

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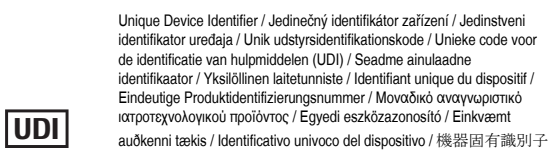
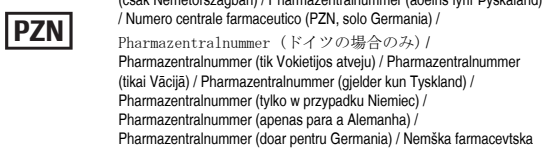
ACCU-CHEK, COBAS, and NAVIFY are trademarks of Roche. INTRALIPID is a trademark of Fresenius Kabi AB.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2024, Roche Diagnostics

0123

Roche Diagnostics GmbH

Sandhofer Strasse 116

68305 Mannheim, Germany

www.roche.com

+800 5505 6606