



INSERT ALIGN DOTS:

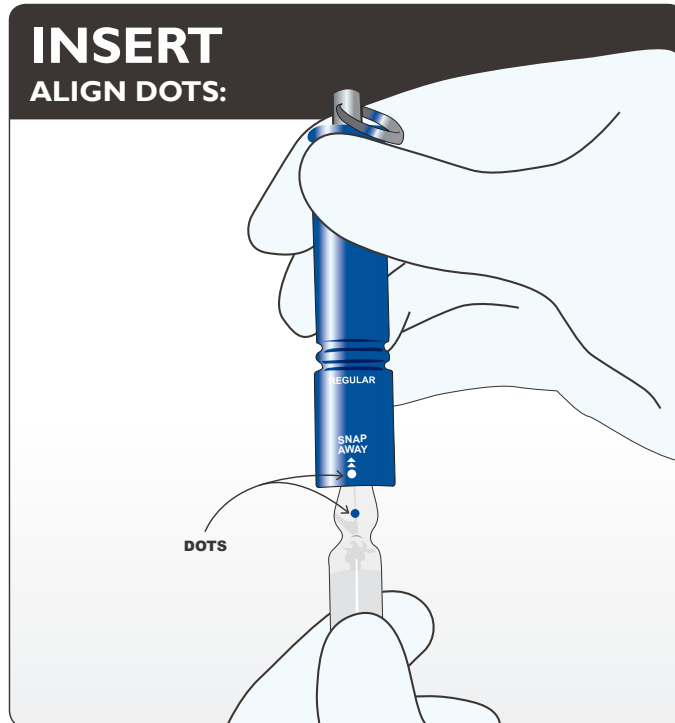


1. Align the dots if present. Gently insert all of the ampoule lid into the SnapIT. The ampoule lid becomes locked in at the neck.

SNAP IT AWAY FROM DOT:

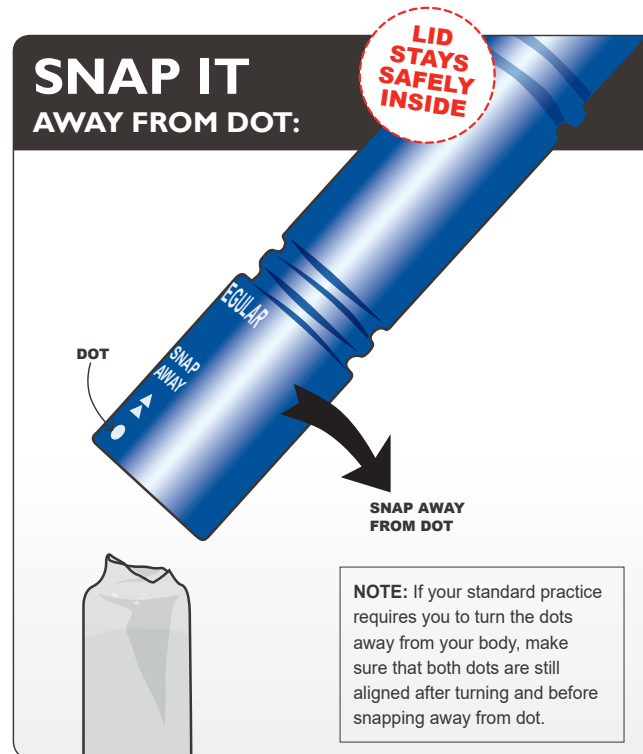


2. With little effort, lever away from the dot to snap off the lid. The lid should remain held in the SnapIT.

EJECT LID IN BIN:

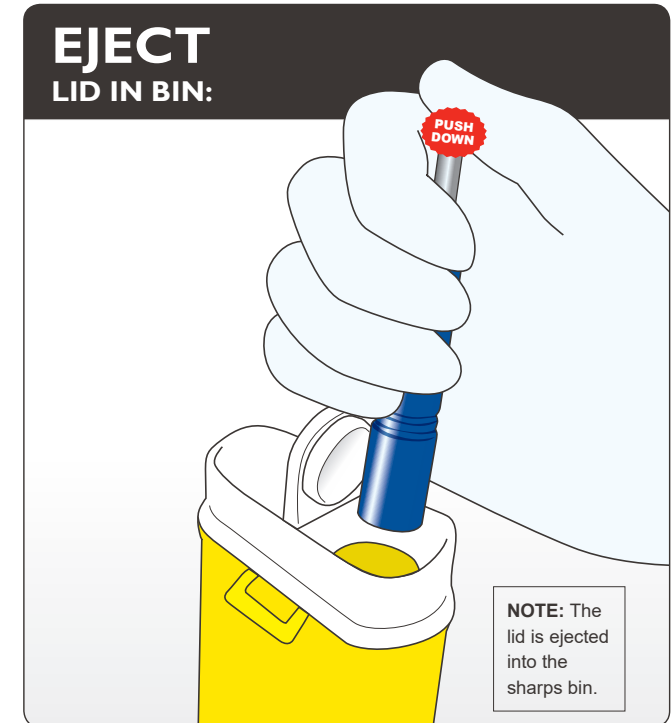

3. Carefully aim over a sharps bin and eject the lid by pushing down on the rod with your thumb.

CLEANING YOUR SNAPIT:

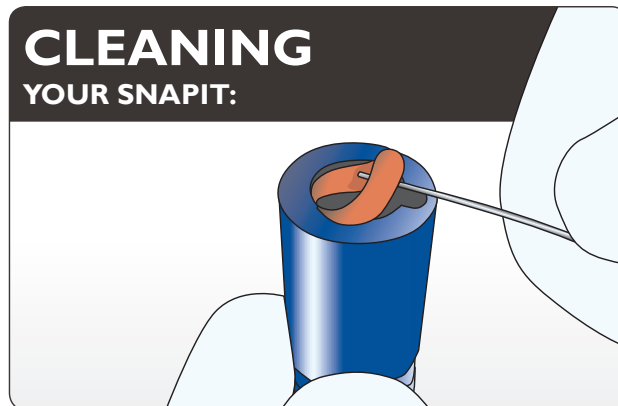

4. Clean your SnapIT regularly. Remove O-ring as shown to disassemble the parts and clean with water or hospital grade cleaner. Do this in a clean dish to avoid loss of parts. Dry completely then re-assemble.

NOTE: If the ampoule lid breaks, tap the SnapIT over a sharps bin to remove excess glass pieces then rinse under running water before disassembling.


SIZES:

If the ampoule lid is too big, it will not fit through the entrance. Use a larger SnapIT.

If the ampoule is too small, the SnapIT will not hold the top securely. Use a smaller SnapIT.


Top Tips: Using the SnapIT® reusable ampoule opener

- 1. Right Size:** Check that you are using the correct size SnapIT for your ampoule.

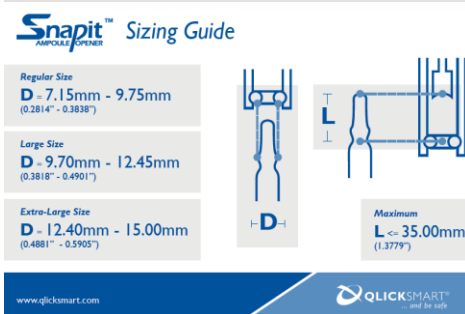

Tip: There is a size guide available for reference.

- 2. Align Dots:** For one-point-cut scored ampoules, align the dot on ampoule with dot on SnapIT.

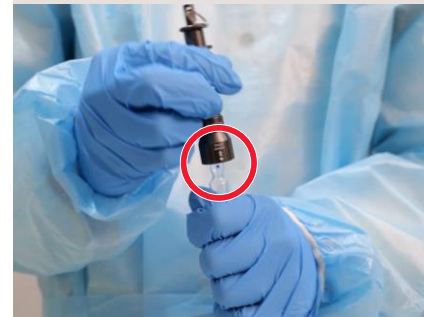


Tip: Full scored ampoules can be aligned in any direction.

- 3. Eject cap:** Press the eject rod above a sharps bin to discard the ampoule head.



Tip: If the rod is extended, it needs to be ejected.

- 4. Cleaning:** Disassemble and clean your SnapIT regularly for optimum use.

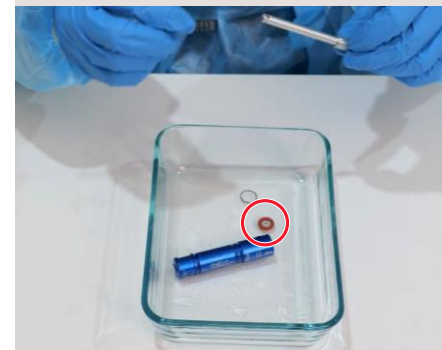

Tip: Check your O-Ring in case it needs to be replaced.



Scan the QR Code to see SnapIT in action. Contact us to arrange sharps safety education for your facility:
w | www.quicksmart.com e | hello@quicksmart.com p | +61 7 3844 1182