Ergänzende Montageanweisung	02	DEU
Additional Installation instructions	03	ENG

D^{med}® Visiano 20-2 C E1

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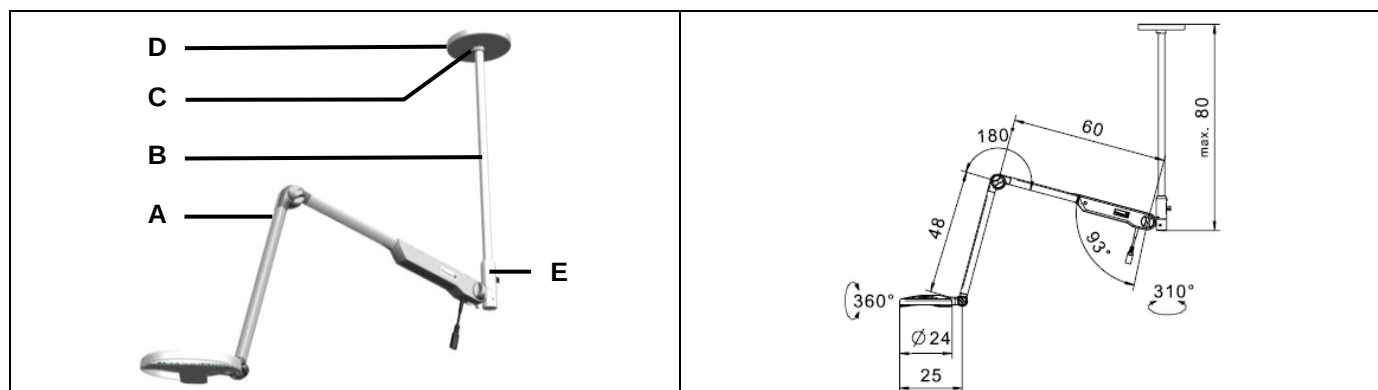
DANGER

Assembly by qualified personnel

- Assembly must be carried out by qualified personnel only. Lack of appropriate knowledge could be life-threatening.

DANGER

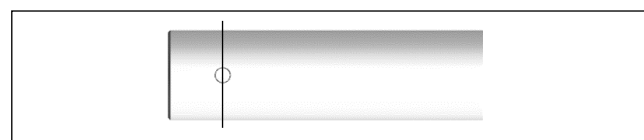
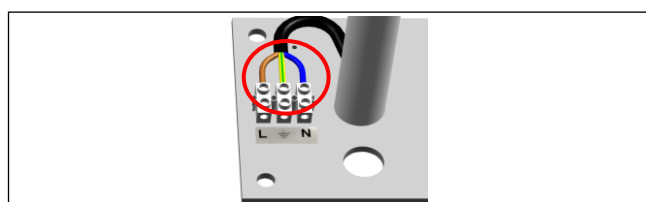
Life-threatening danger from falling light.

- The ceiling must be made of solid concrete to guarantee a secure hold.


DANGER

Electric shocks are life-threatening


- It must be possible to separate the light from the mains by an external all-pole switch (not included).
- Voltage supply from the power connection must be protected on all poles by means of an overcurrent release (according to national installation requirements; it is not included in the scope of supply).

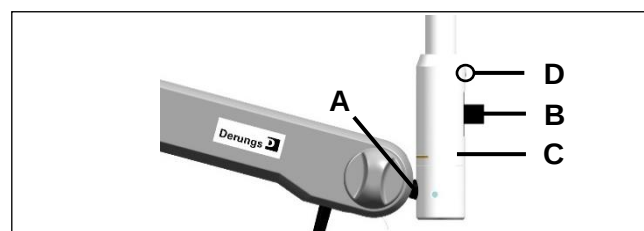- maximum ceiling height is 2800mm
- The lowest point of the standard ceiling pipe (after installation) should not be deeper than 2000mm.
- Remove the power cable before shortening ceiling tube
- Shorten the ceiling tube to the desired length with a metal saw and deburr.

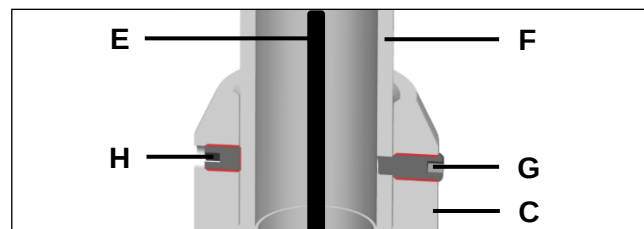- Place the connection attachment (C) on the vertical tube.
- Align the pin attachment (A) centrally in the desired, future working range of the light.
- The cable connection (B) must be positioned exactly opposite the pin holder (A).
- Mark the hole to be drilled in the ceiling tube through hole (D) in the connection attachment and remove the attachment.
- Drill a 4mm hole in the vertical tube.

- Insert the power cable (E) into the ceiling conduit and connect it to the luminaire terminal.
- Slide the connection attachment (C) back onto tube (F). Turn until the drilled hole and the existing hole of the attachment overlap. Screw in the hexagon socket screw (G) here.
- Screw the slotted screw (H) flush into the opposite hole

OWN DISTRIBUTORS

GERMANY

Derungs Medical GmbH
Rudolf-Diesel-Strasse 2
78239 Rielasingen-Worblingen
Germany
Telephone +49 (0) 7731 909719-0
Mail: info@derungsmedical.com

ITALY

Waldmann Illuminotecnica S.r.l.
Via della Pace, 18 A
20098 San Giuliano Milanese (MI)
Italy
Telephone +39 02 98 24 90 24
Fax +39 02 98 24 63 78
www.waldmann.com
info-it@waldmann.com

GERMANY

Herbert Waldmann GmbH & Co. KG
Postfach 5062
78057 Villingen-Schwenningen
Germany
Telephone +49 7720 601 0
Telephone +49 7720 601 100 (Sales)
Fax +49 7720 601 290
www.waldmann.com
sales.germany@waldmann.com

NETHERLANDS

Waldmann BV
Lingewei 19
4004 LK Tiel
Netherlands
Telephone +31 344 631 019
Fax +31 344 627 856
www.waldmann.com
info-nl@waldmann.com

SWITZERLAND

Waldmann Lichttechnik GmbH
Benkenstrasse 57
5024 Küttigen
Switzerland
Telephone +41 62 839 12 12
Fax +41 62 839 12 99
www.waldmann.com
info-ch@waldmann.com

SWEDEN

Waldmann Ljusteknik AB
Skebokvarnsvägen 370
124 50 Bandhagen
Sweden
Telephone +46 8 990 350
Fax +46 8 991 609
www.waldmann.com
info-se@waldmann.com

AUSTRIA

Waldmann Lichttechnik Ges.m.b.H
Gewerbepark Wagram 7
4061 Pasching/Linz
Austria
Telephone +43 7229 67 400
Fax +43 7229 67 444
www.waldmann.com
info-at@waldmann.com

USA

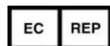
Waldmann Lighting Company
9, W. Century Drive
Wheeling, Illinois 60090
USA
Telephone +1 847 520 1060
Fax +1 847 520 1730
www.waldmannlighting.com
waldmann@waldmannlighting.com

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© Derungs Licht AG • D80.174.000 • 12.2022 • Index: 1.2
Waldmann Eclairage S.A.S
Derungs Licht AG